






Auscultation Trainer App Update Guide

What you need:

- Smart device with outdated Auscultation Trainer App
- Computer
- Smart device charging cord
- New .apk file (found in email from Realityworks Product Support)
- RealityScope

Uninstall the outdated app:




1. Download the .apk file onto your computer. You can find the file at this link: [http://update.realityworks.com/HostUpdates.php?dev=AUS&p=A&l=en&subsys\[HST\]=1.0.0.0](http://update.realityworks.com/HostUpdates.php?dev=AUS&p=A&l=en&subsys[HST]=1.0.0.0)
2. Power on the smart device and tap the **Auscultation Trainer App** icon to open the app.
3. Tap the **side menu** icon and select **Manage Devices**.
4. Select the paired **RealityScope** and tap **Forget Device**.
5. Exit the app.
6. Press and hold the Auscultation Trainer App icon and drag it to the **Uninstall** option at the top of the screen.
7. Tap **OK** on the popup that asks if you want to uninstall the app.

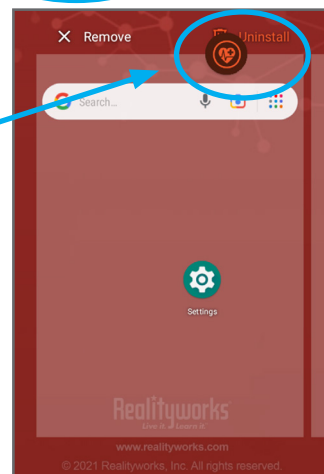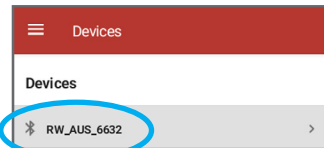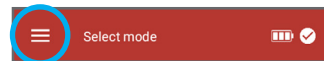

AUSCULTATION TRAINER APP UPDATE GUIDE

Install the latest app:


1. Plug the smart device into your computer using the charging cord.
2. Swipe down from the top of the screen to open the **Top Menu** and tap the **Charging This Device** option.
3. Select **Media Device** from the connection options. You can now see the smart device under Devices and Drives in your computer files.

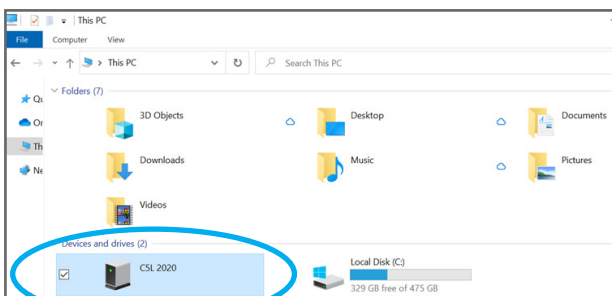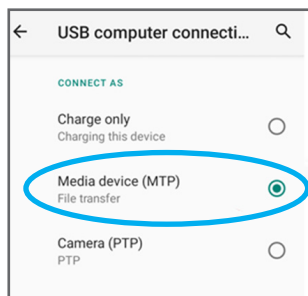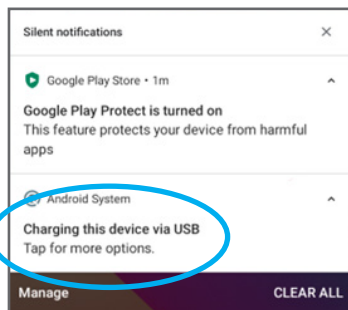


4. Copy the .apk file from your Downloads folder into the Download folder on your smart device. To find the smart device's Download folder, open the smart device's files from the Devices and Drives section of your computer files and open the Internal Storage folder.
5. After the file is copied over, unplug the smart device from your computer.

Set up the latest app:


1. Swipe up from the bottom of the smart device's screen to open the App List.
2. Open the **Files** app and select **Downloads**.

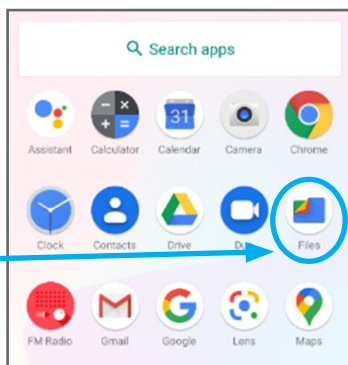


3. Find and select the .apk file with today's date.

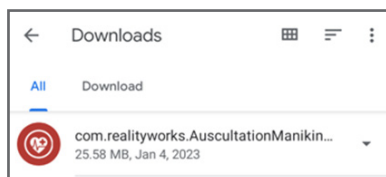

4. Tap **Install** on the popup that appears and wait for the app to install.

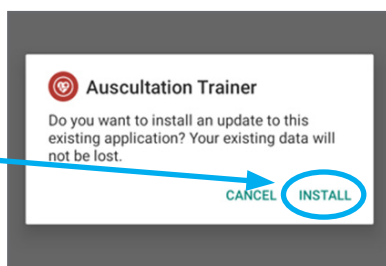


5. Once the app is installed, tap **Done** and return to the main menu.

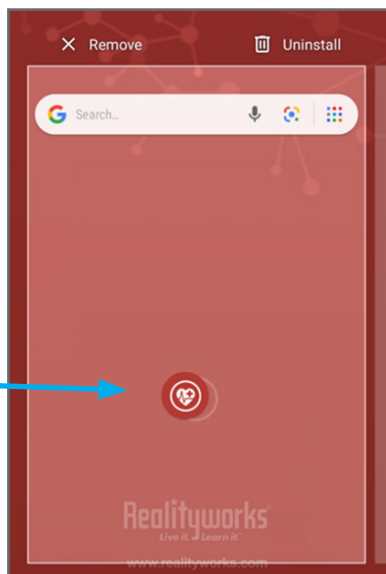

6. Press and hold the Auscultation Trainer App icon, drag it to the left, and center it on the main screen where it was originally.

7. Tap the **Auscultation Trainer App** icon to open it and tap **Allow Only While Using the App** when asked about device location access.

NOTE: Location permission is required by applications for the app to scan for Bluetooth devices, like the *RealityScope*.

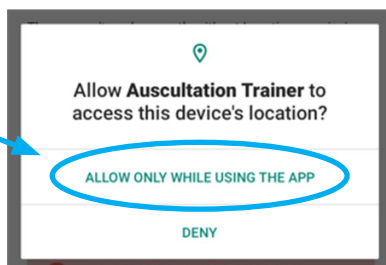


AUSCULTATION TRAINER APP UPDATE GUIDE

Set up the latest app:

8. Pair the **RealityScope** to the updated app by selecting the **Search for New Devices** option at the bottom of the screen.
9. Press the Bluetooth pairing button on the **RealityScope**. The pairing light will flash.
10. Once the **RealityScope** is found, tap Connect.
11. Tap **Accept** on the two popups that appear.
12. After the **RealityScope** is connected, tap Cancel and use the scope as normal or pair more scopes.

NOTE: You can pair up to **5 RealityScopes** with the Auscultation Trainer App.

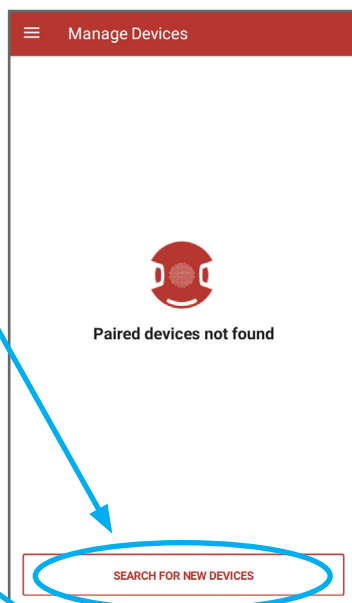

Additional help and information

Product Support

We're here to help! If you have further questions, contact product support at 1.800.830.1416 or productsupport@realityworks.com