

Is my simulator functioning correctly?

Purpose

Troubleshoot issue when no lights or incorrect lights illuminate on the Upper Arm Intramuscular Injection Simulator.

Steps to Perform

1. Print out the Upper Arm Intramuscular Injection site template located on the second page of this document.
2. Cut out the injection site template.
3. Power on the simulator.
4. Locate the Acromion site on the arm and place the Acromion tab on the template over that position. Leave the rest of the cutout to hang down over the outside of the arm.
5. Insert 1" needle at 90-degree angle at each of the 3 injection sites.

Test Location Indicators:

1. Correct site GREEN LED. If too deep displays additional RED LED w/Buzzer
2. Incorrect site RED LED + RED Nerve LEDs + Buzzer
3. Correct site GREEN LED. If too deep displays additional RED LED w/Buzzer

*If no lights appear contact product support.

For additional assistance and video tutorials visit our website:


www.realityworks.com/support/

Upper Arm Intramuscular Injection site cutout

PART #:1048228-01

DIRECTIONS:

- Locate Acromion.
- Position Acromion tab over Acromion position.
- Test at 1, 2, 3 locations.

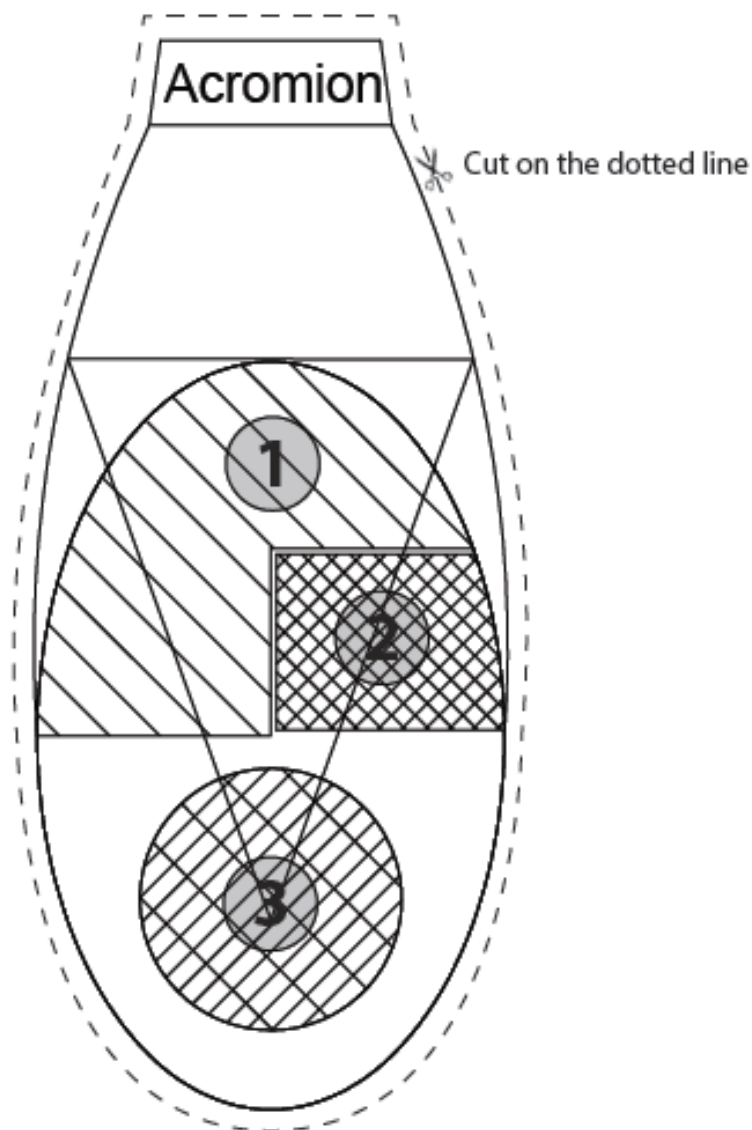

1 = Correct site GREEN LED. If too deep displays additional RED LED w/Buzzer

2 = Incorrect site RED LED + RED Nerve LEDs + Buzzer

3 = Correct site GREEN LED. If too deep displays additional RED LED w/Buzzer